MFR# OMX06U1

Reclining Lift Chair

Chaise de Levage Inclinable

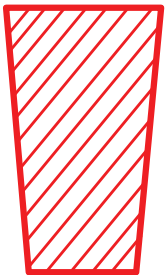
Silla de Elevación Reclinable

# Assembly Instructions

## Instructions d'assemblage

## Instrucciones de ensamble

**IMPORTANT, RETAIN FOR FUTURE REFERENCE: READ CAREFULLY**  
For more information, visit: [www.northridge-home.com](http://www.northridge-home.com)

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For parts or customer service call: **1-866-387-8065**

Or email: [info@northridge-home.com](mailto:info@northridge-home.com)

(English, French and Spanish languages spoken)

Monday-Friday 8am - 8pm (EST).

Telephone number is only valid in the United States and Canada.

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
Telefono no válido en Mexico.

**WARNING: Small parts choking hazard**

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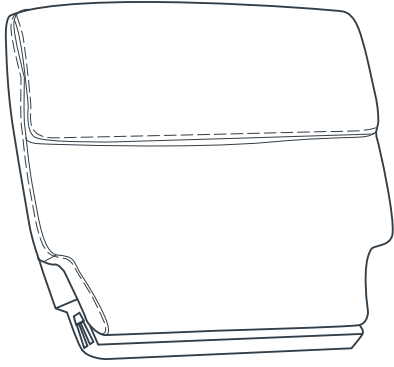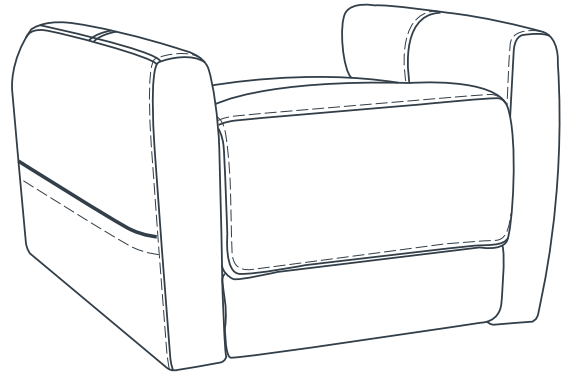
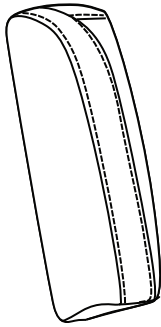
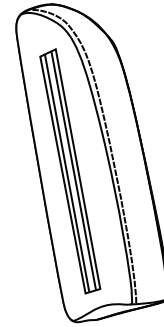
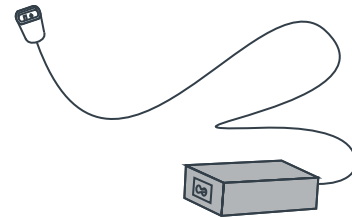
F x 1

### Caution

Do not allow children to play on this piece of furniture, or operate the mechanism. Leg rest folds down upon closing, possibly causing injury. Always leave in an upright and closed position and keep hands and feet clear of mechanism. Only the occupant should operate it.

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# IMPORTANT SAFETY INSTRUCTIONS:

## READ ALL INSTRUCTIONS BEFORE USING

**WARNING: To reduce the risk of burns, fire, electric shock, or injury to persons:**

1. DANGER: To reduce the risk of electric shock - Always unplug this furnishing from the electrical outlet before cleaning.
2. Unplug from outlet before putting on or taking parts off.
3. Close supervision is necessary when used by or near children, or disabled people.
4. Use this furnishing only for its intended use as described in these instructions. Do not use attachments not recommended by the manufacturer.
5. Never operate this furnishing if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged or dropped into water.
6. Keep the cord away from heated surfaces.
7. Never drop or insert any object into any openings.
8. Do not use outdoors. Just for household indoor use only.
9. Do not operate where aerosol (spray) products are being used or where oxygen is being administered.
10. To disconnect, turn all controls to the off position, then remove plug from outlet.
11. WARNING: Risk of injury - Keep children away from extended foot support.
12. WARNING: Risk of injury to Persons - do not use this furnishing to support video equipment such as televisions or computer monitors.
13. Maximum weight limit not to exceed more than 400 pounds.
14. When not in use, it is recommended that the wand be stored securely in the pocket on the outside of the arm.

## SAVE THESE INSTRUCTIONS: SERVICING OF DOUBLE-INSULATED PRODUCTS.

In a double-insulated product, two systems of insulation are provided instead of grounding. No grounding means is provided on a double-insulated product, nor is a means for grounding to be added to the product. Servicing a double-insulated product requires extreme care and knowledge of the system, and is to be done only by qualified service personnel. Replacement parts for a double-insulated product must be identical to the parts they replace.

**Note:** This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: 1) this device may not cause interference, and 2) this device must accept any interference, including interference that may cause undesired operation. Use of this product may be prohibited as a result of any modification made without the consent of the party responsible for compliance.

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# TROUBLE SHOOTING

PROBLEM	POSSIBLE FACTORS	SOLUTIONS
<p>Back will not recline up or down</p>	<ul style="list-style-type: none"> <li>- Power supply is not connected to socket</li> <li>- Wiring plugs are not connected correctly according to the figure on page 4</li> </ul>	<ul style="list-style-type: none"> <li>- Check the power supply has been correctly connected to the socket</li> <li>- Confirm that wiring plugs are connected correctly</li> </ul>
<p>Leg rest will not move up or down</p>	<ul style="list-style-type: none"> <li>- Power supply is not connected to socket</li> <li>- Wiring plugs are not connected correctly according to the figure on page 4</li> </ul>	<ul style="list-style-type: none"> <li>- Check the power supply has been correctly connected to the socket</li> <li>- Confirm that wiring plugs are connected correctly</li> </ul>
<p>Chair doesn't recline or lift</p>	<ul style="list-style-type: none"> <li>- Transformer may not be connected well</li> </ul>	<ul style="list-style-type: none"> <li>- Check the power supply has been correctly connected to the socket</li> <li>-Confirm the wire from the transformer is firmly connected.</li> </ul>
		<p>If after making sure all connections have been properly made and the functions of the chair still do not perform, try unplugging the chair from the outlet for ten minutes. Then replug the transformer into the power outlet and try again.</p>

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## Care & Cleaning:

- Avoid exposing furniture directly to sunlight or heating and air conditioning outlets.
- Exposure to sunlight or heating and air conditioning outlets will fade/damage cover.
- The micro suede fabric covering on this specific piece of furniture is soft to the touch and very durable for every day living. It is recommended by the fabric manufacturer to only clean this type of fabric with water-based cleaning agents or foam. Lightly scrubbing the fabric will help with any tougher spots that may have developed. Always check a spot of fabric in an area that is unseen to be sure the cleaning agent being used will not discolor or fade the fabric's finish. Let air dry and vacuum for best results.

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Costco Wholesale Corporation
Importadora Primex S.A. de C.V.